

NPPTL COVID-19 Response: International Respirator Assessment

Manufacturer: Dongguan Mau Gee Industrial Ltd.

Model Tested: KN95 Disposable Face Mask

Date Tested: May 7, 2020

These findings pertain to the Dongguan Mau Gee Industrial Ltd., KN95 Disposable Face Mask. The packaging for this product indicates that it meets GB2626-2006 (the Chinese standard for Respiratory Protective Equipment – Non-Powered Air-Purifying Particle Respirator).

Ten respirators were submitted for evaluation. The samples were tested using a modified version of NIOSH Standard Test Procedure (STP) TEB-APR-STP-0059. This modified assessment plan can be found [here](#).

No certificate of approval was provided with the samples received; therefore, the authenticity of the claims cannot be validated.

The maximum and minimum filter efficiency was 97.24% and 88.43%, respectively. Eight respirators measured more than 95%. Two respirators measured less than 95%.

While the above-listed product classification has similar performance requirements to NIOSH-approved devices, NIOSH does not have knowledge about the sustained manufacturer quality system and product quality control for these products. NIOSH also does not have knowledge about the product's handling and exposures after leaving its manufacturer's control.

In addition, this product is an ear loop design. Currently, there are no NIOSH-approved products with ear loops; NIOSH-approved N95s have head bands. Furthermore, limited assessment of ear loop designs, indicate difficulty achieving a proper fit. While filter efficiency shows how well the filter media performs, users must ensure a proper fit is achieved.

This assessment is not a part of the NIOSH respirator approval process and will in no way lead to or preclude NIOSH approval through the official approval process. This assessment was developed as an assessment of the filter efficiency for those respirator's represented as certified by an international certification authority, other than NIOSH, to support the availability of respiratory protection to US healthcare workers due to the respirator shortage associated with COVID-19. Only particulate filter efficiency was assessed.

The results provided in this letter are specific to the subset of samples that were provided to NPPTL for evaluation.

These results will be used to update the CDC guidance for [Crisis Capacity Strategies \(during known shortages\)](#).

Evaluation of International Respirators

Test: Modified TEB-APR-STP-0059

Date Tested: May 7, 2020

Report Prepared: May 8, 2020

Manufacturer: Dongguan Mau Gee Industrial Ltd.

Item Tested: KN95 Disposable Face Mask

Country of Certification: China (GB2626-2006)

Pictures have been added to the end of this report.

Filter	Flow Rate (Lpm)	Initial Filter Resistance (mmH ₂ O)	Initial Percent Leakage (%)	Maximum Percent Leakage (%)	Filter Efficiency
1	85	12.9	2.88	2.88	97.12
2	85	11.0	3.12	3.12	96.88
3	85	9.4	4.73	4.73	95.27
4	85	13.3	11.57	11.57	88.43
5	85	12.9	4.11	4.11	95.89
6	85	8.5	5.16	5.16	94.84
7	85	9.9	4.59	4.59	95.41
8	85	12.0	3.03	3.03	96.97
9	85	12.2	2.76	2.76	97.24
10	85	11.4	4.84	4.84	95.16
Minimum Filter Efficiency: 88.43			Maximum Filter Efficiency: 97.24		

- The test method utilized in this assessment is not the NIOSH standard test procedure that is used for certification of respirators. Respirators assessed to this modified test plan do not meet the requirements of STP-0059, and therefore cannot be considered equivalent to N95 respirators that were tested to STP-0059.
- Respirators tested may not be representative of all respirators with the same certification mark. NIOSH has no control over suppliers and distributors of respirators certified by other national or international parties.
- This assessment is not a confirmation that it conforms with any or all of its specifications in accordance with its certification mark.
- This assessment was not a part of the NIOSH approval program. These results do not imply nor preclude a future approval through the NIOSH respirator approval program.

NPPTL COVID-19 Response: International Respirator Assessment

CERTIFICATE OF CONFORMITY 合格证	
Product Name: 产品名称	Disposable Face Mask 一次性使用防护口罩
Product No.: 产品型号	KN95
Product Specification: 产品规格	16CMX10.5CM 检验专用章
Standard Practice 执行标准	GB2626-2006
Production Registration Company 生产/注册企业	Dongguan Mau Gee Industrial Ltd. 东莞茂志针织有限公司
Production Registration Address 生产/注册地址	Xiangtou, Dalang, Dongguan, Guangdong, China 东莞市大朗镇巷头小区富康路 338号茂荣集团工业园
Inspectors: 质检员	01
Serial Number: 生产批号	200405
Production Date: 生产日期	2020/4/5
Validity Period: 有效期	2 years 2年
Quantity: 数量	2500Pcs/Box

NPPTL COVID-19 Response: International Respirator Assessment



NPPTL COVID-19 Response: International Respirator Assessment



NPPTL COVID-19 Response: International Respirator Assessment

