

NPPTL COVID-19 Response: International Respirator Assessment

Manufacturer: 3M

Model Tested: 9542

Date Tested: October 7, 2020

These findings pertain to the 3M, model 9542. The packaging and labeling for this product indicate that it meets GB2626-2006 (the Chinese standard for Respiratory Protective Equipment – Non-Powered Air-Purifying Particle Respirator).

Ten respirators were submitted for evaluation. The samples were tested using a modified version of NIOSH Standard Test Procedure (STP) TEB-APR-STP-0059. This modified assessment plan can be found [here](#).

No certificate of approval was provided with the samples received; therefore, the authenticity of the claims cannot be validated.

The maximum and minimum filter efficiency was 99.87% and 99.73%, respectively. All ten respirators measured more than 95%.

While the above-listed product classification has similar performance requirements to NIOSH-approved devices, NIOSH does not have knowledge about the sustained manufacturer quality system and product quality control for these products. NIOSH also does not have knowledge about the product's handling and exposures after leaving its manufacturer's control.

This product has head bands/straps. While filter efficiency shows how well the filter media performs, users must ensure a proper fit is achieved.

This assessment is not a part of the NIOSH respirator approval process and will in no way lead to or preclude NIOSH approval through the official approval process. This assessment was developed as an assessment of the filter efficiency for those respirators represented as certified by an international certification authority, other than NIOSH, to support the availability of respiratory protection to US healthcare workers due to the respirator shortage associated with COVID-19. Only particulate filter efficiency was assessed.

The results provided in this letter are specific to the subset of samples that were provided to NPPTL for evaluation.

These results will be used to update the CDC guidance for [Crisis Capacity Strategies \(during known shortages\)](#).

Evaluation of International Respirators

Test: Modified TEB-APR-STP-0059

Date Tested: October 7, 2020

Report Prepared: October 9, 2020

Manufacturer: 3M

Item Tested: 9542

Country of Certification: China (GB2626-2006)


Pictures have been added to the end of this report.

Filter	Flow Rate (LPM)	Initial Filter Resistance (mmH ₂ O)	Initial Percent Leakage (%)	Maximum Percent Leakage (%)	Filter Efficiency (%)
1	85	12.8	0.07	0.26	99.74
2	85	13.0	0.13	0.21	99.79
3	85	11.4	0.06	0.14	99.86
4	85	13.0	0.08	0.25	99.75
5	85	12.1	0.05	0.16	99.84
6	85	9.8	0.03	0.17	99.83
7	85	12.2	0.04	0.27	99.73
8	85	12.4	0.08	0.13	99.87
9	85	11.1	0.05	0.18	99.82
10	85	10.7	0.06	0.19	99.81
Minimum Filter Efficiency: 99.73%			Maximum Filter Efficiency: 99.87%		

- The test method utilized in this assessment is not the NIOSH standard test procedure that is used for certification of respirators. Respirators assessed to this modified test plan do not meet the requirements of STP-0059, and therefore cannot be considered equivalent to N95 respirators that were tested to STP-0059.
- Respirators tested may not be representative of all respirators with the same certification mark. NIOSH has no control over suppliers and distributors of respirators certified by other national or international parties.
- This assessment is not a confirmation that it conforms with any or all of its specifications in accordance with its certification mark.
- This assessment was not a part of the NIOSH approval program. These results do not imply nor preclude a future approval through the NIOSH respirator approval program.


NPPTL COVID-19 Response: International Respirator Assessment






警告

本产品只用于对某些颗粒物的呼吸防护。错误使用可导致疾病甚至死亡。欲了解正确使用方法，请参见包装上说明，请示您的主管或致电3M公司，400 920 3595。



06-31-201001



WARNING

This respirator helps protect against certain particles. Misuse can result in sickness or death. For proper use, see supervisor or User Instructions on package or call 3M, 400-920-3595.

产品符合：
GB 2626-2006 KN95标准

This respirator confirms:
GB 2626-2006 KN95

XX003823257

3M中国有限公司 上海市田林路222号
3M China co.Ltd., 222 Tian Lin Road

邮编：200233
P.R. China(200233)

电话：86-21-2210 5335
Tel: 86-21-2210 5335


NPPTL COVID-19 Response: International Respirator Assessment



NPPTL COVID-19 Response: International Respirator Assessment



NPPTL COVID-19 Response: International Respirator Assessment

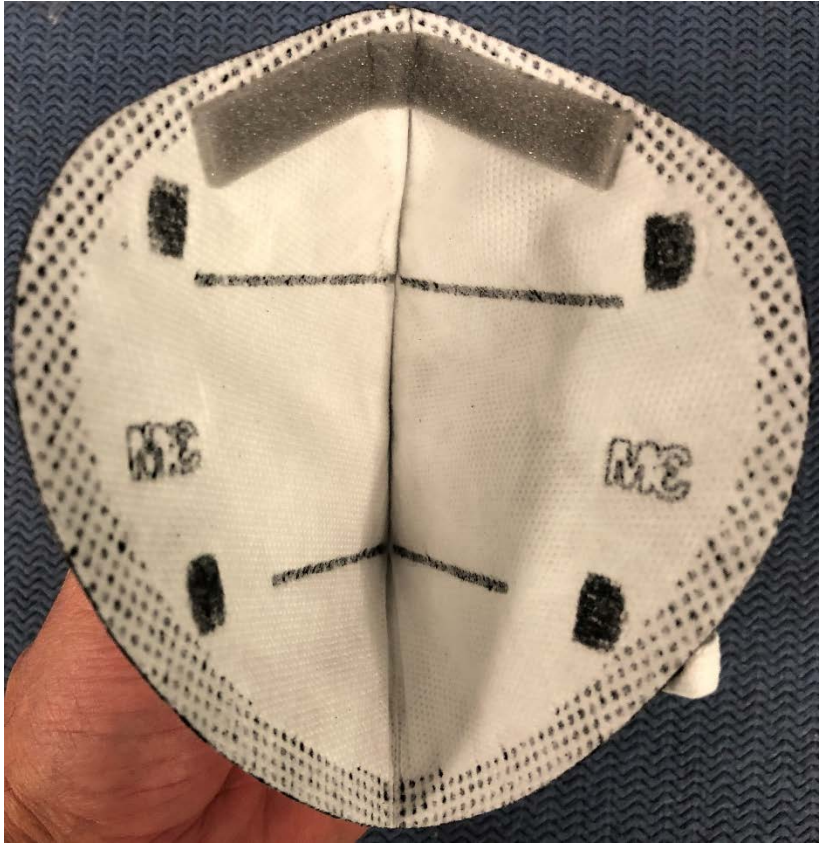

NPPTL COVID-19 Response: International Respirator Assessment

